

COVID-19 Antigen Rapid Test Kit


(Colloidal Gold)




Simple | Accurate | Rapid

Xiamen AmonMed Biotechnology Co., Ltd.


Operating Procedure

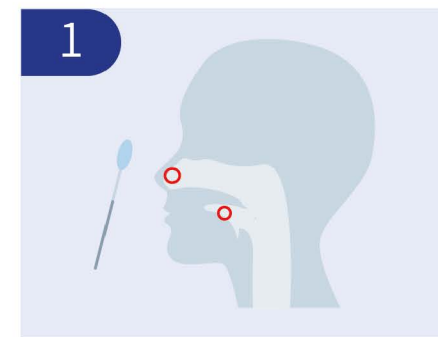

1 Collect throat/nasal specimen with swab.

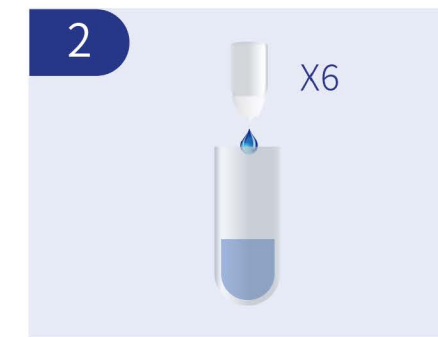


2 Apply 6 drops of antigen extraction solution to the extraction tube.

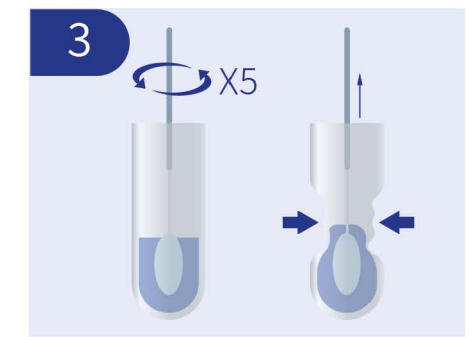


3 Insert the swab into extraction tube, swirl the swab at least five times. Remove the swab while squeezing the sides of the tube to extract the liquid from the swab.



4 Press the nozzle cap tightly onto the tube.

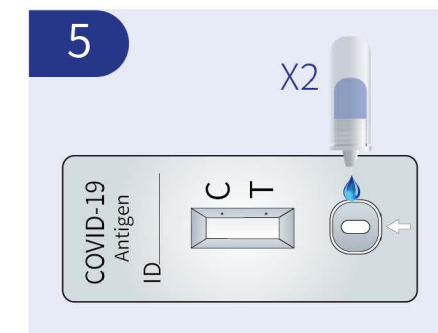


5 Apply 2 drops of extracted specimen to the specimen well of the test card.

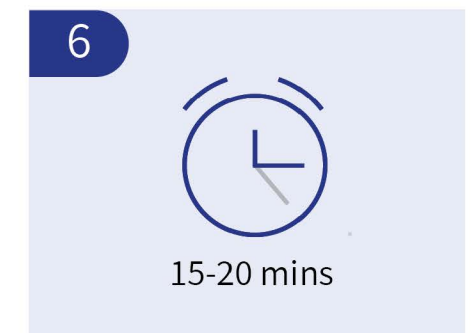

6 Read the test result at 15–20 minutes.

Interpretation of Test Result

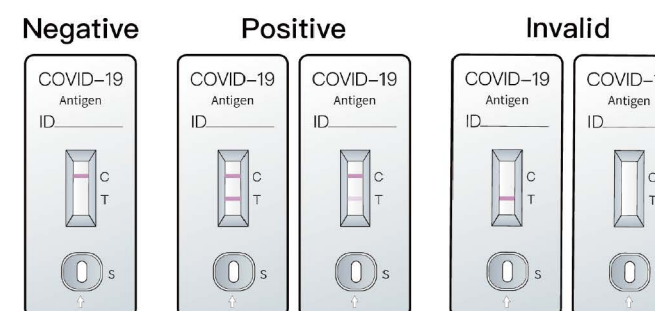

Clinical Performance

Relative Sensitivity: 95.05% (88.93%~97.87%)
 Relative Specificity: 99.55% (97.47%~99.92%)
 Accuracy: 98.13% (95.98%~99.14%)

* 95% Confidence Interval