

# COVID-19 IgM/IgG Test Kit


(Colloidal Gold)




**Simple | Accurate | Rapid**

**Xiamen AmonMed Biotechnology Co., Ltd.**


## Operating Procedure

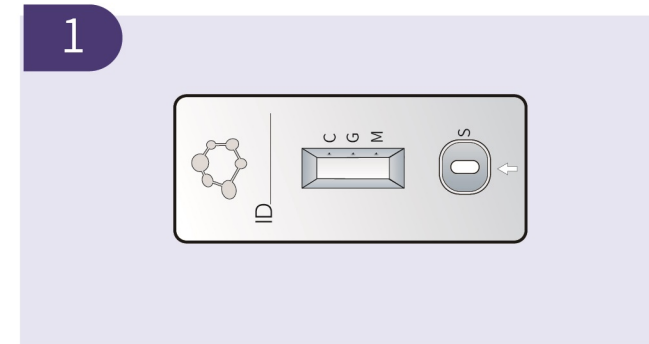


Take the test card to the room temperature.

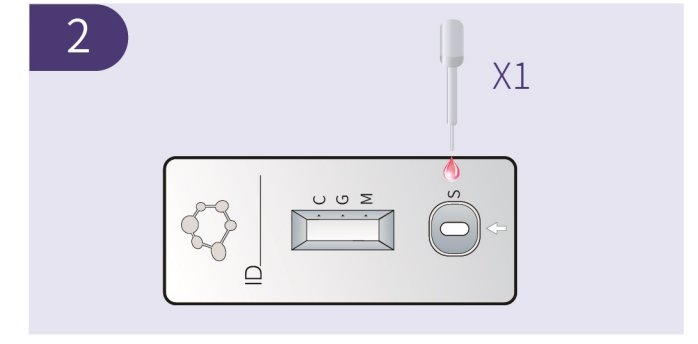

Apply 1 drop of serum/plasma (10 µL), or whole blood (15 µL).

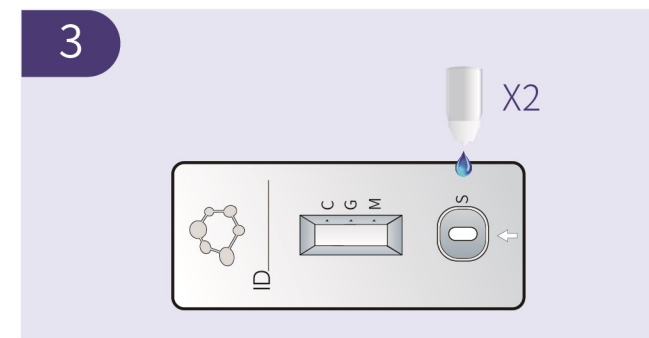


Apply 2 drops of diluents (about 80 µL) to the specimen well.

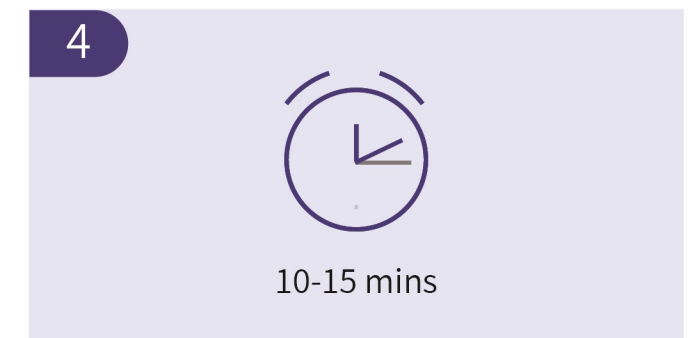


Interpret the result after 10 minutes but no later than 15 minutes.

## Interpretation of Test Result


**Negative**

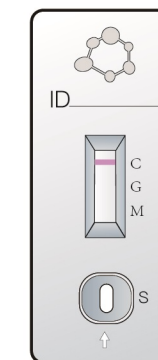

**Positive**

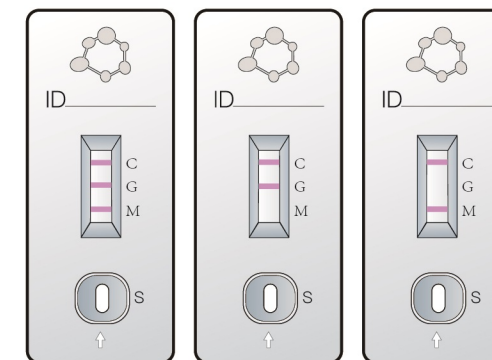

**Invalid**

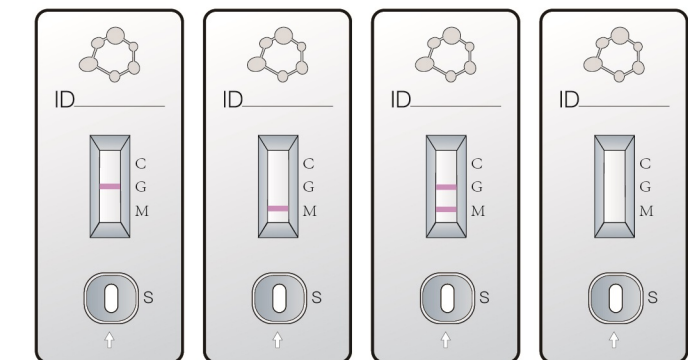

	IgM	IgG	IgM/IgG
Sensitivity	88% (22/25)	98% (49/50)	98% (49/50)
Specificity	100% (50/50)	100% (50/50)	100% (50/50)

\* 95% Confidence Interval