


COVID-19 Antigen Rapid Test Kit (Colloidal Gold) for Anterior Nasal Swab



Simple | Accurate | Rapid
For Private Use

Principle

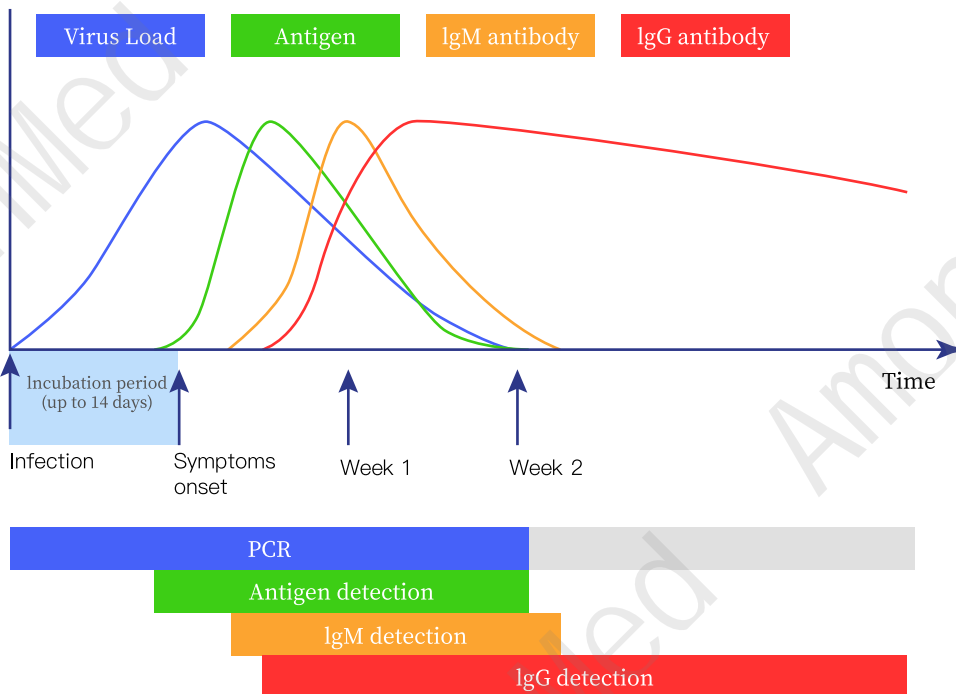

COVID-19 Antigen Rapid Test Kit (Colloidal Gold) employs immuno-chromatography technology to detect the novel corona virus antigen in human nasal specimens.

A colored test line (T) would be visible in the result window, if SARS-CoV-2 antigens are present in the specimen. Absence of the test line (T) suggests a negative result.

DISCLAIMER: This is for information purposes only. Detection windows may vary from person to person.

Xiamen AmonMed Biotechnology Co., Ltd.


Tel: +86 592 6519307

<http://www.amonmed.com>


E-mail: info@amonmed.com

Address: Unit 503, 120 Xinyuan Road, Haicang District, Xiamen, Fujian, China


Operating Procedure

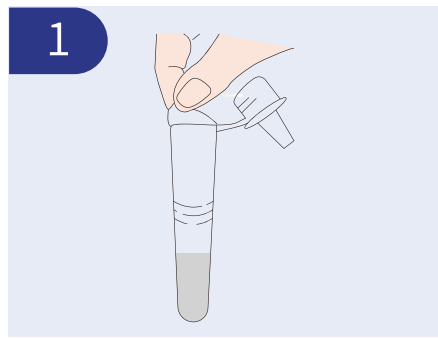


1 Peel off the foil seal from the top of extraction tube, being sure to keep the extraction tube upright.

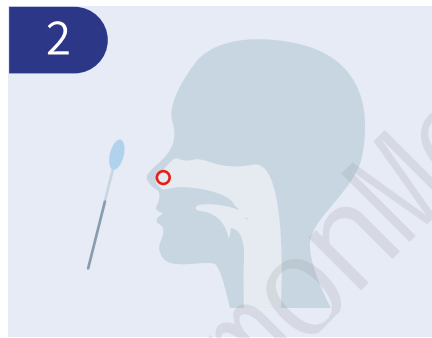


2 Collect nasal specimen with a swab.

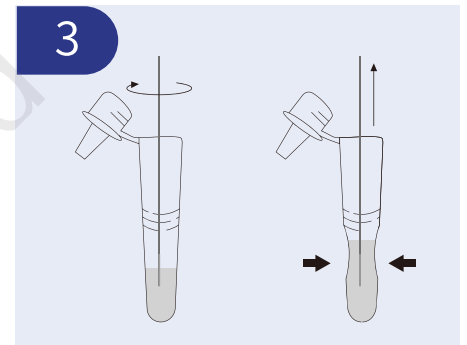


3 Insert the nasal swab into extraction tube. Stir the nasal swab more than 5 times. Leave nasal swab in extraction tube for about 1 minute. Squeeze the swab against the inner wall of extraction tube to release the liquid as much as possible when you remove the swab.

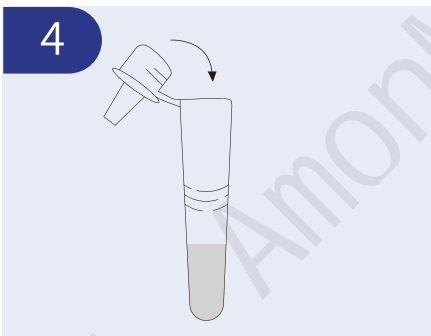

4 Press the cap onto the extraction tube tightly.

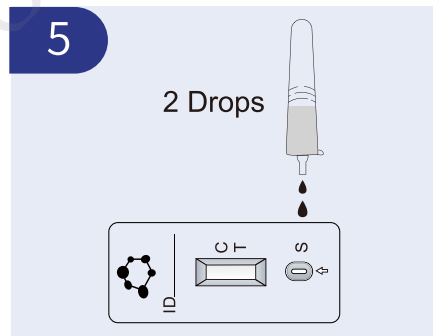


5 Apply 2 drops of extracted specimen to the specimen well of the test card.

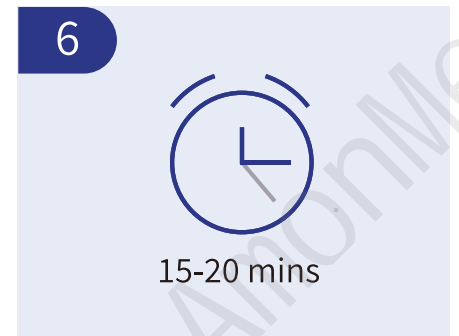


6 Read the test result at 15–20 minutes.

Interpretation of Test Result


Negative

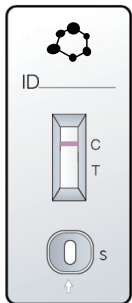

Positive

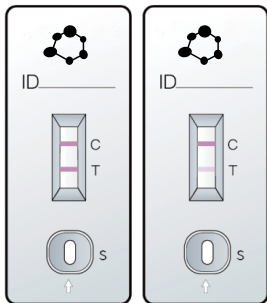

Invalid

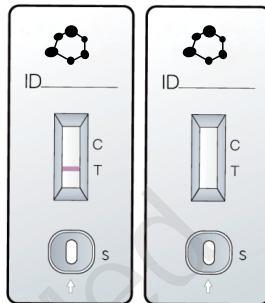

Product Name	Contents	Packing	Sample Type	Catalog No.	Shelf Life
COVID-19 Antigen Rapid Test Kit (Colloidal Gold)	Nasal Swab Extraction Tube with Extraction Solution Test Card Instructions for Use Tube Rack	1 Test/Kit 5 Tests/Kit 25 Tests/Kit 50 Tests/Kit	Nasal Swab	CG01Ag-01N-ST CG01Ag-05N-ST CG01Ag-25N-ST CG01Ag-50N-ST	18 Months